

# Models: LSD, LSP, LSS

## Equipoise™ Clinical Lighting

### User Manual



**With care and attention to the following information your Equipoise Lamp will give you years of satisfactory service.**

#### **IMPORTANT:**

- Take care when unpacking the lamp and removing the Twist tie while the sprung frame opens out.
- The Lamp Head has an anti-rotation Stop. Do not forcibly turn the head in one direction when the stop is encountered.

#### **Mounting:**

- For Equipoise lamps used on a heavy table base chose a location that is level.
- For Lamps used on a Table Clamp or Wall Bracket ensure the base is mounted vertical.
- Correctly mounting your Equipoise Lamp ensures that it is stable and does not swing away from the desired task. Alternative Bases can be purchased to mount your lamp in the perfect position.
- This product is manufactured to AS 60598.2.4:2019 and AS/NZS 60598.1:2017+A1 and is rated as LED Only 19 watt maximum to prevent fire damage when in contact with materials or injury to persons.

#### **CAUTION:**

**If the external Cable or cord of this Luminaire is damaged, it shall be exclusively replaced by the manufacturer or his service agent or a similar qualified person in order to avoid a hazard.**

**Do not use outdoors nor in wet areas.**

#### **Optional Mounting Bases:**

	LSM 2	LSM 3H	LSM 4	LSM 11	LSM 14	LSM 16
LSD	•	•	•	•	•	•
LSP	•	•	•	•	•	•
LSS	•	•	•	•	•	•

#### **LED Lamps Available:**

##### **Model LSD (E27)**

- GLSC-6EWW, GLS-9EWW

##### **Model LSP (E27)**

- PAR38-HPLED 24° Spot, 5200Lux@1Metre
- SBL-18EWW 90° Beam Angle, 500Lux@1Metre

##### **Model LSS (E27)**

- L302-9W, 15° Reflector Lamp